



New
Instruments


Maintenance of Instruments

- ! - Keep instruments open for cleaning!
- Rinse with distilled water!
- ! - Ensure sufficient drying time!



Operating Room

- ! - Intended use!
- Apply the instruments for wet or dry handling!

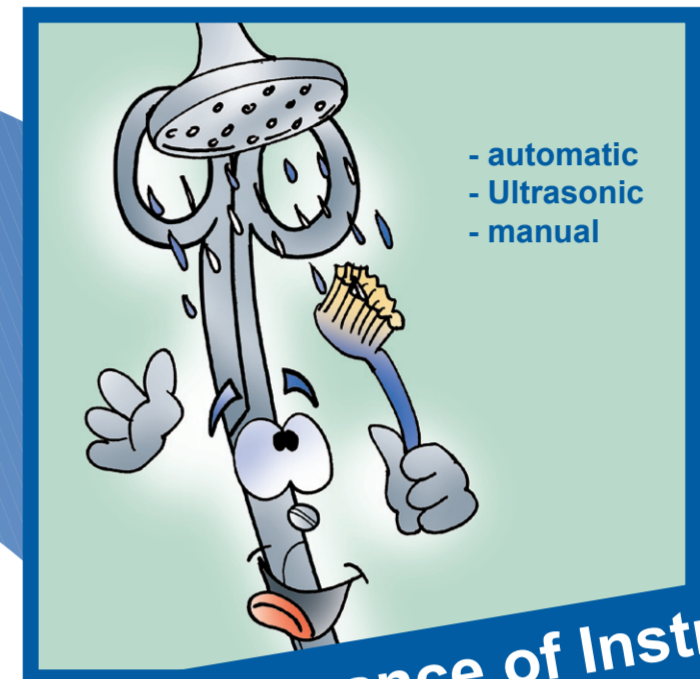

Maintenance of Instruments

- automatic
- Ultrasonic
- manual


Steam sterilization



- ! - Use only distilled water!
- $\geq 134^{\circ}\text{C}$!
- Respect norms!


Lawton's
**Lucky
Lawrence**

Sort and pack

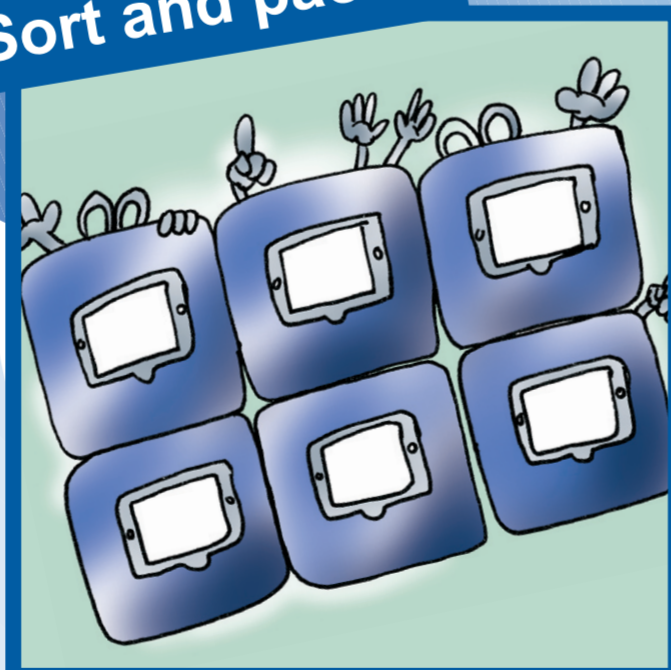

- ! - Open ratchet!
- Don't overload trays!

Visual check and functional test

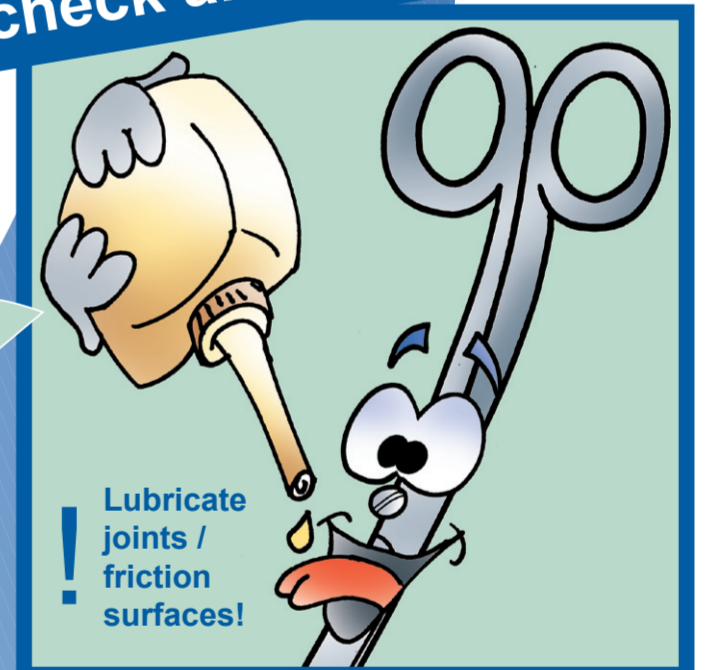

- ! Lubricate joints / friction surfaces!

- ! Handle special instruments separately!

Defective
Instruments



Please order our brief
handbook - „Maintenance of
Instruments“ for free!

www.lawton.de

Your manufacturer:

LAWTON GmbH & Co. KG
Württembergische Straße 23
78567 Fridingen/Germany
Tel. +49 (0) 7463 9965-0

